Material included in the package:

GM501 Cult
Product code: 4 GM 501H-20
1 x 20 ml GM501 Cult with Gentamicin
Product code: 4 GM 501H-50
1 x 50 ml GM501 Cult

Material included in the package:

GM501 Cult with Gentamicin
Product code: 4 GM 501H+G-20
1 x 20 ml GM501 Cult with Gentamicin
Product code: 4 GM 501H+G-50
1 x 50 ml GM501 Cult with Gentamicin

Material included in the package:

GM501 Cult with Gentamicin and Phenolred
Product code: 4 GM 501H+PR+G-20
1 x 20 ml GM501 Cult with Gentamicin and Phenolred
Product code: 4 GM 501H+PR+G-50
1 x 50 ml GM501 Cult with Gentamicin and Phenolred

Material not included in the package:

- CO₂-incubator at 37°C
- Petri-dishes
- LAF bench (ISO 5 environment)
- Microscope
- Test tubes

Product specifications and quality control:

- All raw materials are of highest available purity (European Pharmacopoeia and/or USP standard) if applicable.
- A certificate of analysis is available for each batch upon request from our website with respect to lot number.
- The MSDS for the media are available upon request and can also be downloaded from our website.
- The media are manufactured and tested according to the following specifications:

| | |
|--|--|
| pH (at 37°C, 6% CO ₂) | 7.20-7.45 |
| Osmolality (mOsm/kg) | 270-290 |
| Sterility | sterile - SAL10 ⁷ (Sterility Assurance Level) |
| Endotoxins (EU/ml) | < 0.25 |
| Mouse Embryo Assay (MEA) (1-cell assay, blastocysts after 96 h in % , 96 h exposure at zygote stage) | ≥80 |

EN: Intended user/intended users:

- The media are ready-to-use bicarbonate-buffered culture media, designed for fertilisation and for human embryo culture from day 1 to blastocyst stage. It can also be used for embryo transfer.

- The intended users are IVF professionals (lab technicians, embryologists or medical doctors).

Content:

GM501 Cult
NaCl, KCl, KH₂PO₄, MgSO₄ 7H₂O, NaHCO₃, CaCl₂ 2H₂O, D(+)-Glucose anhydrous, Na-Lactate (50% solution), Na-Pyruvate, EDTA, Alaninyl-Glutamine, water, non-essential and essential Amino Acids, human serum albumin (10,00 mg/ml), Phenolred.

GM501 Cult with Gentamicin
NaCl, KCl, KH₂PO₄, MgSO₄ 7H₂O, NaHCO₃, CaCl₂ 2H₂O, D(+)-Glucose anhydrous, Na-Lactate (50% solution), Na-Pyruvate, EDTA, Alaninyl-Glutamine, water, non-essential and essential Amino Acids, human serum albumin (10,00 mg/ml), Gentamicin (10,00 mg/ml).

GM501 Cult with Gentamicin and Phenolred
NaCl, KCl, KH₂PO₄, MgSO₄ 7H₂O, NaHCO₃, CaCl₂ 2H₂O, D(+)-Glucose anhydrous, Na-Lactate (50% solution), Na-Pyruvate, EDTA, Alaninyl-Glutamine, water, non-essential and essential Amino Acids, human serum albumin (10,00 mg/ml), Gentamicin (10,00 mg/ml), Phenolred.

Instructions for use:

- The medium must be equilibrated over night in a humidified CO₂-incubator (at 5-7 % CO₂, 37°C).
- The media are designed for embryo culture under oil as well as in open systems.

Precautions and warnings:

- GM501 Cult with Gentamicin and GM501 Cult with Gentamicin and Phenolred contains the antibiotic Gentamicin Sulfate. Appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic.

- Standard measures to prevent infections resulting from the use of medicinal products prepared from human blood or plasma include selection of donors, screening of individual donations and plasma pools for specific markers of infection and the inclusion of effective manufacturing steps for the inactivation/removal of viruses. Despite this, when medicinal products prepared from human blood or plasma are administered, the possibility of transmitting infective agents cannot totally be excluded. This also applies to unknown or emerging viruses and other pathogens. There are no reports of proven virus transmissions with albumin, manufactured to European Pharmacopoeia specifications by established processes.

- Therefore, handle all specimens as if capable of transmitting HIV or hepatitis. Always wear protective clothing when handling specimens. Always work under strict hygienic conditions (ISO 5 environment, e.g. LAF-bench) to avoid possible contamination, even when GM501 Cult medium contains Gentamicin.

- For the intended use.
- The user facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

Pre-checks:

- Do not use the product if it becomes discoloured, cloudy, or shows any evidence of microbial contamination.
- Do not use the product if seal of the container is opened or defective when the product is delivered.

Storage instructions and stability:

- The shelf life is 6 month from time of manufacture.
- Store between 2-8 °C.
- Do not freeze before use.
- Keep away from (sun) light.
- The product can be used safely up to 7 days after opening, when sterile conditions are maintained and the product is stored at 2-8 °C.
- Stable after transport (max. 5 days) at elevated temperature (≤ 37°C).
- Content cannot be re-sterilized after opening.
- Do not use after expiry date.

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GM501 Cult with Gentamicin and Phenolred

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GYNEMED GmbH & Co. KG