# PrioCHECK™ Neospora Internal Reference Serum/Milk

Catalog Number RNEO

Doc. Part No. 100019959 Pub. No. MAN0008613 Rev. B.0

For use with:

PrioCHECK™ Bovine Neospora Ab 2.0 Serum/Milk Kit (Cat. Nos. LSINE0A2, LSINE0A5)

 **WARNING!** Read the Safety Data Sheets (SDSs) and follow the handling instructions. Wear appropriate protective eyewear, clothing, and gloves. Safety Data Sheets (SDSs) are available from [thermofisher.com/support](http://thermofisher.com/support).

 **WARNING! POTENTIAL BIOHAZARD.** Read the biological hazard safety information at this product's page at [thermofisher.com](http://thermofisher.com). Wear appropriate protective eyewear, clothing, and gloves.

## Product description

Applied Biosystems™ PrioCHECK™ Neospora Internal Reference Serum/Milk is an antiserum of known status for use as an internal reference sample.

## Contents and storage

Component	Volume (100 tests)	Storage
Internal Reference Neo	500 µL	Below -16°C

## Guidelines for use

- We recommend storage of the internal reference in small aliquots to avoid more than 5 freeze-thaw cycles.
- For use with serum samples, test the reference **as a sample**.
- For use with individual milk samples, dilute the reference **1:20** in 2b - Milk Negative C. Neo reagent (provided in the PrioCHECK™ Bovine Neospora Ab 2.0 Serum/Milk Kit). Test the diluted reference **as a sample**.
- For use with **pools of milk** samples, dilute the reference **1:100** in 2b - Milk Negative C. Neo reagent (provided in the PrioCHECK™ Bovine Neospora Ab 2.0 Serum/Milk Kit). Test the diluted reference **as a sample**.
- Each laboratory must determine the frequency of use. We recommend, however, using an internal reference for each experiment.

## Documentation and support

### Customer and technical support

Technical support: visit [thermofisher.com/askaquestion](http://thermofisher.com/askaquestion)  
Visit [thermofisher.com/support](http://thermofisher.com/support) for the latest in services and support, including:

- Worldwide contact telephone numbers
- Order and web support
- User guides, manuals, and protocols
- Certificates of Analysis
- Safety Data Sheets (SDSs; also known as MSDSs)  
**NOTE:** For SDSs for reagents and chemicals from other manufacturers, contact the manufacturer.

### Limited product warranty

Life Technologies Corporation and/or its affiliate(s) warrant their products as set forth in the Life Technologies' General Terms and Conditions of Sale found on Life Technologies' website at [www.thermofisher.com/us/en/home/global/terms-and-conditions.html](http://www.thermofisher.com/us/en/home/global/terms-and-conditions.html). If you have any questions, please contact Life Technologies at [thermofisher.com/support](http://thermofisher.com/support).

 Prionics Lelystad B.V. | Platinestraat 33 | 8211 AR Lelystad | The Netherlands

The information in this guide is subject to change without notice.

**DISCLAIMER:** TO THE EXTENT ALLOWED BY LAW, LIFE TECHNOLOGIES AND/OR ITS AFFILIATE(S) WILL NOT BE LIABLE FOR SPECIAL, INCIDENTAL, INDIRECT, PUNITIVE, MULTIPLE, OR CONSEQUENTIAL DAMAGES IN CONNECTION WITH OR ARISING FROM THIS DOCUMENT, INCLUDING YOUR USE OF IT.

Revision history of Pub. No. MAN0008613 (English/French)

Revision	Date	Description
B.0	9 August 2017	Updated to the current document template, with associated updates to the warranty, trademarks, and logos.
A.0	14 April 2014	Baseline for revision history

**Limited Use Label License No. 473: Veterinary Diagnostic Assay.** The purchase of this product conveys to the purchaser the limited, non-transferable right to use the purchased amount of the product only to perform veterinary diagnostic services including reporting the results of such services for a fee, and internal research for the sole benefit of the purchaser. No right to resell, repackage, or distribute this product, or any of its components, is conveyed expressly, by implication, or by estoppel. For information on obtaining additional rights, please contact [outlicensing@lifetech.com](mailto:outlicensing@lifetech.com) or Out Licensing, Life Technologies, 5791 Van Allen Way, Carlsbad, California 92008.

©2017 Thermo Fisher Scientific Inc. All rights reserved. All trademarks are the property of Thermo Fisher Scientific and its subsidiaries unless otherwise specified.

## For Laboratory Use.

[thermofisher.com/support](http://thermofisher.com/support) | [thermofisher.com/askaquestion](http://thermofisher.com/askaquestion)

[thermofisher.com](http://thermofisher.com)

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## Documentation et support

### Service clientèle et assistance technique

Support technique : rendez-vous sur [thermofisher.com/askaquestion](http://thermofisher.com/askaquestion)

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Prionics Lelystad B.V. | Platinastraat 33 | 8211 AR Lelystad | The Netherlands