CERTEST FOB+Transferrin

ONE STEP human haemoglobin
+ human transferrin COMBO
CARD TEST

CERTEST BIOTEC S.L.

For information purposes only

For information purposes only

ENGLISH**INTENDED USE**

CerTest FOB+Transferrin one step combo card test is a coloured chromatographic immunoassay (cut-off qualitative test) for the simultaneous qualitative determination of human haemoglobin (hHb) and human transferrin (hTf) in stool samples.

CerTest FOB+Transferrin combo card test offers a simple, a highly sensitive and non-invasive screening assay to make a presumptive diagnosis of gastrointestinal bleeding.

INTRODUCTION

Colorectal cancer is the second leading cause of illness and death in the Western world. The screening using faecal occult blood tests is based on the bleeding detection, as important symptom of colonic neoplasm, either at an early-stage cancer or large adenomatous polyps. Colorectal cancer is also associated with local acute inflammatory reaction being visualized, in some cases, by white cell neutrophil scanning.

Haemoglobin is the iron-containing oxygen-transport protein in the red blood cells of all vertebrates that may be leaked into gastrointestinal tract and then discharged with the faeces in gastrointestinal bleeding diseases.

Transferrin is a blood-derived component that may be leaked into gastrointestinal tract and then discharged with the faeces in gastrointestinal bleeding diseases. Transferrin is stable in faeces and a good marker to detect loss of blood from the upper and lower intestine (gastrointestinal bleeding).

When gastrointestinal blood is lost, the stool will contain a combination of intact or nearly intact haemoglobin, intact heme, and heme-derived porphyrins in amounts that depend on the site, the amount of bleeding and the transit time through the gut. Immunochemical tests detect intact or nearly intact human haemoglobin, being a very specific technique to detect the loss of blood from the lower intestine, because blood from lower sites is less degraded during transit. The haemoglobin is unstable in faeces, leading to false negative results. The detection of faecal transferrin, which is more stable than haemoglobin, provides an alternative way of diagnosing the disease in the upper digestive tract.

TEST PRINCIPLE

CerTest FOB+Transferrin is based on the principle of a cut-off qualitative immunochromatographic assay for the determination of human haemoglobin and human transferrin in stool samples.

Strip A consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against human haemoglobin and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-human haemoglobin) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming coloured conjugate complexes.

Strip B consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against human transferrin and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-human transferrin) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming coloured conjugate complexes.

If the sample is human haemoglobin positive, the antigens of the diluted sample react with the red-coloured conjugate complex (anti-human haemoglobin monoclonal antibodies-red polystyrene microspheres) in the strip A, and if the sample is human transferrin positive, the antigens of the diluted sample react with the red-coloured conjugated complex (anti-human transferrin monoclonal antibodies-red polystyrene microspheres) in strip B, which were previously pre-dried on the absorbent pad. The mixture then moves upward on the membrane by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-human haemoglobin antibodies present on the membrane of strip A (test line) and the anti-human transferrin antibodies present on the membrane of strip B (test line) capture the coloured conjugate and the red line will be visible in both strips. These bands are used to interpret the result.

If the sample is negative, there is no human haemoglobin and human transferrin presence and yet, the human haemoglobin and the human transferrin may be present in a concentration lower than the cut-off value, for which the reaction will not take place with any red-coloured conjugate complex. The anti-human haemoglobin and anti-human transferrin antibodies present on the membranes (test lines) will not capture the antigen-red-coloured conjugate complex (not formed), for which the red lines will not appear.

Whether the sample is positive or not, in both strips, the mixture continues to move across the membranes to the immobilized specific antibodies placed in the control lines. The anti-specific protein antibodies present on both membranes will capture control green-conjugate complex and both control lines will always appear. The presence of these green lines serve as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents.

STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The test is stable until the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

PRECAUTIONS


- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contamination errors.
- The tests should be discarded in a proper biohazard container after testing.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provided in the kit are approved for use in the CerTest FOB+Transferrin test. Do not use any other commercial kit component.
- The presence of yellow lines in the results window (control line zone and test line zone), before using the test, is completely normal and does not imply failure of the test functionality.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.

SPECIMEN COLLECTION AND PREPARATION

Stool samples should be collected in clean containers. The samples can be stored in the refrigerator (2-8°C) for 7 days prior to testing. For longer storage, maximum 6 months, the specimen must be kept frozen at -20°C. In that case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool sample as thoroughly as possible prior to preparation.

Specimen preparation (see illustration):

1. Take out the cap 1 of the stool collection tube and use the stick to pick up sufficient sample quantity. Introduce the stick once into 4 different parts of the stool sample (1), making sure that, in each insertion, the stick's screw is covered with the sample to take the appropriate amount of stool. Then, introduce the sample in the stool collection tube (2). For liquid samples, add approx. 20µL in the stool collection tube using a micropipette.
2. Close the tube with the diluent and stool sample (2). Shake the tube in order to assure good sample dispersion (3). The stool collection tube with dilute sample can be stored until 2 days at room temperature or until 5 days in the refrigerator (2-8°C) prior to testing.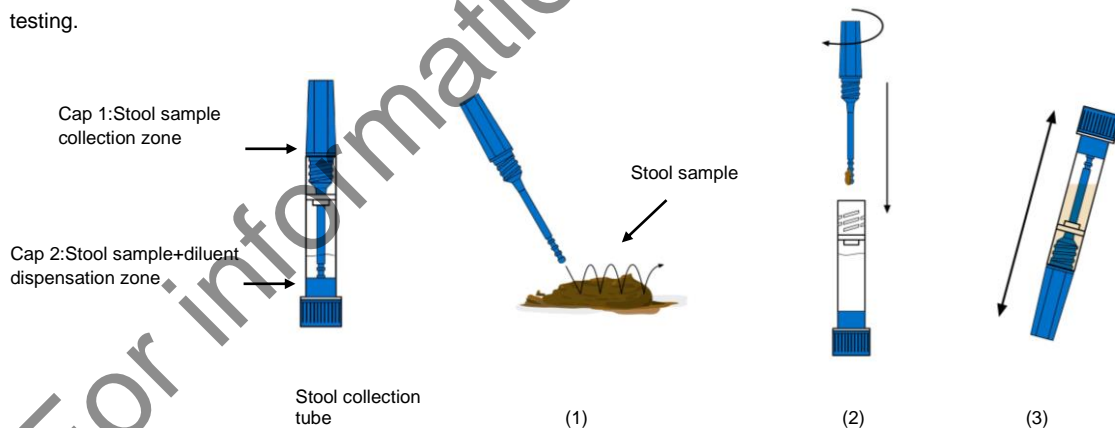

Precautions: Patients should not collect samples during their menstrual period if they have bleeding haemorrhoids, blood in urine or if they have strained during bowel movement.

MATERIALS

MATERIALS PROVIDED

- CerTest FOB+Transferrin combo card tests
- Instructions for use
- Stool collection tubes with diluent

MATERIALS REQUIRED BUT NO PROVIDED



- Specimen collection container
- Disposable gloves
- Timer

TEST PROCEDURE


Allow tests, stool samples and controls to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Proceed to shake the stool collection tube in order to assure good sample dispersion.
2. Remove the CerTest FOB 50+200 combo card test from its sealed bag just before using it.
3. Take the stool collection tube, open the cap 2 (4) and dispense 3 drops in the circular window marked with the letter A (5) and 3 drops, using the same tube, in the circular window marked with the letter B (6). Avoid adding solid particles with the liquid.
4. **Read the result at 10 minutes.** Do not read the test result after more than 10 minutes.


If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.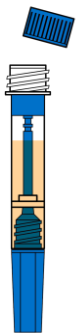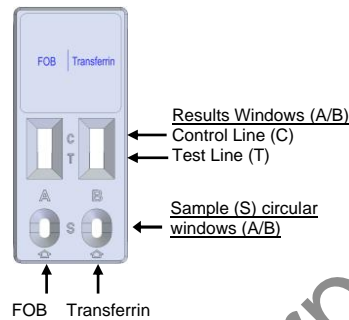

(4)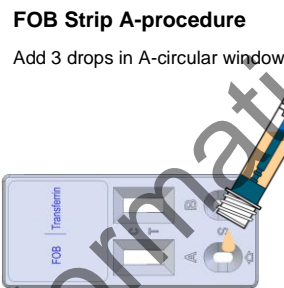

(5)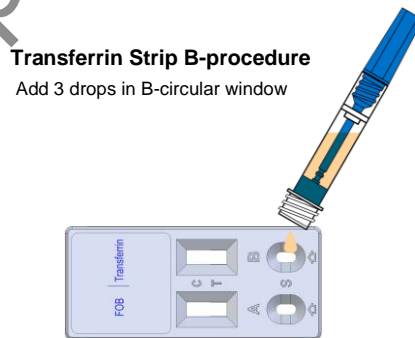

(6)

INTERPRETATION OF THE RESULTS (please refer to the illustration below)

1.


A: Green → Negative hHb
B: Green → Negative hTf
2.

A: Green/Red → Positive hHb
B: Green/Red → Positive hTf
3.

A: Green/Red → Positive hHb
B: Green → Negative hTf
4.

A: Green → Negative hHb
B: Green/Red → Positive hTf

INVALID RESULTS

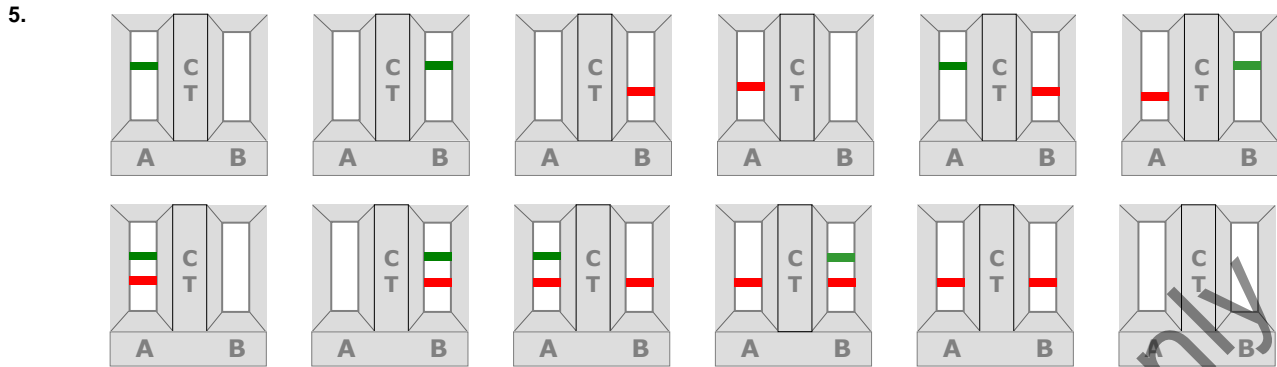	A (FOB (hHb))	B (Transferrin (hTf))	Interpretation of the results
1.	- GREEN	- GREEN	There is no human haemoglobin and human transferrin presence. Haemoglobin and transferrin markers are not present in patient sample (<50ng/mL for haemoglobin and <4ng/mL for transferrin), which might mean no faecal occult blood and no gastrointestinal bleeding.
2.	+ GREEN-RED	+ GREEN-RED	There is human haemoglobin and human transferrin presence. That probably would mean a lower gastrointestinal bleeding disease (colorectal cancer). However, it should not discriminate upper gastrointestinal disease.
3.	+ GREEN-RED	- GREEN	There is human haemoglobin presence. That probably would mean a lower gastrointestinal bleeding disease (the concentration of human transferrin in blood is 100 times less than human haemoglobin, therefore, hHb positive means not much blood present in faeces).
4.	- GREEN	+ GREEN-RED	There is human transferrin presence. That probably would mean an upper gastrointestinal bleeding disease (the human haemoglobin was probably degraded in the gastrointestinal tract).
5.	Any other result	Any other result	Invalid result, either A or B, we recommend repeating the assay using the same sample with another combo card test.

INVALID: Total absence of any control coloured line (GREEN) regardless the appearance or not of the result lines (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control lines failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persist, discontinue using the test kit and contact your local distributor.

NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured bands in the test lines (T) in the results windows will vary depending on the concentration of human haemoglobin and human transferrin present in the specimen.

QUALITY CONTROL

Internal procedural controls are included in the test. The green lines appearing in the control lines (C) in the results windows are internal controls, which confirm sufficient specimen volume and correct procedural technique.

LIMITATIONS

1. The test must be carried out within 2 hours after opening the sealed bag.
2. An excess of sample could cause wrong results (brown bands appear). Dilute the sample with the diluent and repeat the test.
3. The intensity of test line may vary from very strong at high human haemoglobin/transferrin concentration to faint when the human haemoglobin/human transferrin concentration is close to the cut-off value of the test.
4. CerTest FOB+Transferrin should be used only with samples from human faeces. The use of other samples has not been established. The quality of the test depends on the quality of the sample; proper faecal specimens must be obtained.
5. Positive results determine the presence of human haemoglobin and/or human transferrin in faecal samples; nevertheless, it can be due to several causes, besides colorectal bleeding, such as haemorrhoids, blood in urine or stomach irritations. A positive result should be followed up with additional diagnostic procedures to determine the exact cause and source of the blood.

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- A negative result is not meaningful because of it is possible the human haemoglobin and/or human transferrin concentration in the stool sample is lower than cut-off values. Negative results do not exclude bleeding, as some polyps and colorectal cancers may bleed intermittently or not during certain stages of the disease. Moreover, blood may not be uniformly distributed in stool samples.
- Patients should not collect samples during their menstrual period, if they have bleeding haemorrhoids, blood in urine or if they have strained during bowel movement.
- Mucous stool samples may cause non-specific reactions in the test. Every mucous stool sample whose result is positive should be followed up with other techniques of diagnosis to confirm the result.
- Faecal samples whose antigen concentration is near to the cut-off value of the CerTest FOB+Transferrin test could be lost if the sample has been diluted and preserved until 2 days at room temperature or until 5 days at 2-8°C until to be tested. For faecal samples near to the detection limit value is better to dilute the sample and make immediately the test.

EXPECTED VALUES

Colorectal cancer affects both men and women of all racial and ethnic groups, being most often found in people aged 50 years or older. Regarding men, colorectal cancer is the third most common cancer after prostate and lung cancers. Regarding women, colorectal cancer is the third most common cancer after breast and lung cancers.

Some studies established equal or higher 5.1µg hHb/g faeces for haemoglobin and 0.4µg hTf/g faeces for transferrin as cut-off values to allow detect adult patients with gastrointestinal bleeding that will require to diagnosis additional diagnostic invasive procedures.

PERFORMANCE CHARACTERISTICS

Analytical sensitivity (detection limit)

Cut-off values of CerTest FOB+Transferrin are 50ng/mL (5.1µg hHb/g faeces) for human haemoglobin and 4ng/mL (0.4µg hTf/g faeces) for human transferrin.

Clinical Sensitivity and Specificity

An evaluation with faecal samples was performed comparing the results obtained by an immunochromatographic test (CerTest FOB+Transferrin, CerTest) and others commercial ELISA tests (Human Hemoglobin ELISA kit, ICL) and (Human Transferrin ELISA Kit, ICL).

The results were as follows:

Data for cut-off value 5 µg hHb/g

		ELISA test: Human Hemoglobin ELISA kit		
		+	-	Total
IC test: CerTest FOB+Transferrin (Strip A: FOB)	+	38	11	49
	-	0	49	49
	Total	38	60	98

CerTest FOB+Transferrin (Strip A: FOB) vs Human Hemoglobin ELISA kit		
	Mean Value	95% confidence interval
Sensitivity	100.0%	90.7– 100.0%
Specificity	81.7%	69.6 – 90.5%
PPV	77.6%	63.4 - 88.2%
NPV	100.0%	92.7 – 100.0%

Specificity for a cut-off value of 1 µg hHb/g: 93.5% (mean value); 82.1-98.6% (95% confidence interval).

Data for cut-off value 0.4 µg hTf/g

		ELISA test: Human Transferrin ELISA kit		
		+	-	Total
IC test: CerTest FOB+Transferrin (Strip B: Transferrin)	+	38	3	41
	-	12	47	59
	Total	50	50	100

CerTest FOB+Transferrin (Strip B: Transferrin) vs Human Transferrin ELISA kit		
	Mean Value	95% confidence interval
Sensitivity	76.0%	61.8 – 86.9%
Specificity	94.0%	83.5 – 98.7%
PPV	92.7%	80.1 - 98.5%
NPV	79.7%	67.2 –98.0%

Sensitivity for a cut-off value of 2 µg hTf/g: 81.1% (mean value); 64.8-92.0% (95% confidence interval).

Specificity for a cut-off value of 0.2 µg hTf/g: 96.3% (mean value); 81.0-99.9% (95% confidence interval).

The results showed a high sensitivity and specificity to detect human haemoglobin and human transferrin using CerTest FOB+Transferrin.

Cross reactivity

An evaluation was performed to determine the cross reactivity of CerTest FOB+Transferrin; no cross reactivity against other gastrointestinal pathogens, other organism, substances and/or faecal markers occasionally present in faeces:

For Strip A: FOB

<i>Adenovirus</i>	<i>Clostridium perfringens</i>	Human lactoferrin	<i>Salmonella typhi</i>
<i>Astrovirus</i>	<i>Cryptosporidium</i>	Human transferrin	<i>Salmonella typhimurium</i>
Bovine haemoglobin	<i>Entamoeba dispar</i>	<i>Legionella pneumophila</i>	<i>Shigella boydii</i>
Bovine transferrin	<i>Entamoeba histolytica</i>	<i>Listeria monocytogenes</i>	<i>Shigella dysenteriae</i>
Bovine lactoferrin	<i>Escherichia coli</i> O111	Norovirus GI	<i>Shigella flexneri</i>
<i>Campylobacter coli</i>	<i>Escherichia coli</i> O26	Norovirus GII	<i>Shigella sonnei</i>
<i>Campylobacter jejuni</i>	<i>Escherichia coli</i> O157	Pig haemoglobin	<i>Streptococcus pneumococcal</i>
<i>Clostridium difficile</i> antigen GDH	<i>Giardia</i>	Rotavirus	<i>Streptococcus pyogenes</i>
<i>Clostridium difficile</i> Toxin A	<i>Helicobacter pylori</i>	<i>Salmonella enteritidis</i>	<i>Yersinia enterocolitica</i> O:3
<i>Clostridium difficile</i> Toxin B	Human calprotectin	<i>Salmonella paratyphi</i> A	<i>Yersinia enterocolitica</i> O:9

For Strip B: Transferrin

<i>Adenovirus</i>	<i>Cryptosporidium</i>	Human lactoferrin	<i>Salmonella typhimurium</i>
<i>Astrovirus</i>	<i>Entamoeba dispar</i>	<i>Legionella pneumophila</i>	<i>Shigella boydii</i>
Bovine haemoglobin	<i>Entamoeba histolytica</i>	<i>Listeria monocytogenes</i>	<i>Shigella dysenteriae</i>
Bovine Transferrin	<i>Escherichia coli</i> O111	Norovirus GI	<i>Shigella flexneri</i>
<i>Campylobacter coli</i>	<i>Escherichia coli</i> O26	Norovirus GII	<i>Shigella sonnei</i>
<i>Campylobacter jejuni</i>	<i>Escherichia coli</i> O157	Pig haemoglobin	<i>Streptococcus pneumococcal</i>
<i>Clostridium difficile</i> antigen GDH	<i>Giardia</i>	Rotavirus	<i>Streptococcus pyogenes</i>
<i>Clostridium difficile</i> Toxin A	<i>Helicobacter pylori</i>	<i>Salmonella enteritidis</i>	<i>Yersinia enterocolitica</i> O:3
<i>Clostridium difficile</i> Toxin B	Human calprotectin	<i>Salmonella paratyphi</i> A	<i>Yersinia enterocolitica</i> O:9
<i>Clostridium perfringens</i>	Human haemoglobin	<i>Salmonella typhi</i>	

No special diet is recommended prior to testing. There are not interferences with any foods (vitamin C, broccoli, carrots...).

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1. Towler BP, Irwig L, Glasziou P, Weller D, Kewenter J. "Screening for colorectal cancer using the faecal occult blood test, Hemocult". Cochrane Database Syst Rev. 2000;(2): CD001216.
2. Ransohoff DF and Lang CA. "Screening for colorectal cancer with the Fecal Occult Blood Test: a background paper. Ann Intern Med". 1997; 126: 811-822.
3. Ransohoff DF and Lang CA. "Suggested technique for Faecal Occult Blood testing and interpretation in colorectal cancer screening". Ann Intern Med. 1997; 126: 808-810.
4. Chien-Hua Chiang et al., "A Comparative Study of Three Fecal Occult Blood Test in Upper Gastrointestinal Bleeding". Kaohsiung J. Med. Sci. May 2006, 22; 223-228.
5. Virtanen et al., "Higher concentrations of serum transferrin receptor in children than in adults", Am. J. Clin. Nutr. 1999; 69: 256-60.

SYMBOLS FOR IVD COMPONENTS AND REAGENTS/

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