CERTEST Transferrin

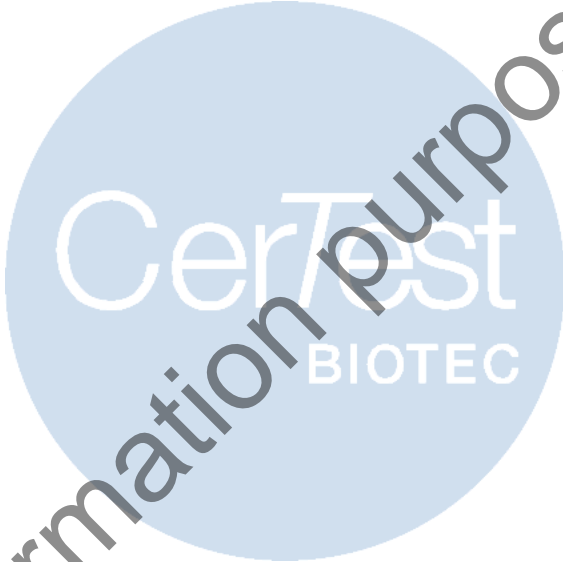
ONE STEP human transferrin
CARD TEST

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ENGLISH

INTENDED USE

CerTest Transferrin one step card test is a coloured chromatographic immunoassay (cut-off qualitative test) for the qualitative determination of human transferrin (hTf) in stool samples.

CerTest Transferrin card test offers a simple, a highly sensitive and non-invasive screening assay to make a presumptive diagnosis of gastrointestinal bleeding.

INTRODUCTION

Colorectal cancer is the second leading cause of illness and death in the Western world. The screening using faecal occult blood tests is based on the bleeding detection, as important symptom of colonic neoplasm, either at an early-stage cancer or large adenomatous polyps. Colorectal cancer is also associated with local acute inflammatory reaction being visualized, in some cases, by white cell neutrophil scanning.

Transferrin is a blood-derived component that may be leaked into gastrointestinal tract and then discharged with the faeces in gastrointestinal bleeding diseases. Transferrin is stable in faeces and a good marker to detect loss of blood from the upper and lower intestine (gastrointestinal bleeding).

When gastrointestinal blood is lost, the stool will contain a combination of intact or nearly intact haemoglobin, intact heme and heme-derived porphyrins in amounts that depend on the site, the amount of bleeding and the transit time through the gut. Immunochemical tests detect intact or nearly intact human haemoglobin, being a very specific technique to detect the loss of blood from the lower intestine, because blood from lower sites is less degraded during transit. The haemoglobin is unstable in faeces, leading to false negative results. The detection of faecal transferrin, which is more stable than haemoglobin, provides an alternative way of diagnosing the disease in the upper digestive tract.

TEST PRINCIPLE

CerTest Transferrin is based on the principle of a cut-off qualitative immunochromatographic assay for the determination of human transferrin in stool samples.

The strip consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test lines (T), in the results window, against human transferrin and with rabbit policlonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with two label solutions: mouse monoclonal antibodies anti-human transferrin conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

If the sample is human transferrin positive, the antigens of the diluted sample react with the red-coloured conjugate complex (anti-human transferrin monoclonal antibodies-red polystyrene microspheres), which were previously pre-dried on the absorbent pad. The mixture then moves upward on the membrane, by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-human transferrin antibodies present on the membrane (test line) capture the coloured conjugates and the red line will be visible. These bands are used to interpret the result.

If the sample is negative, there is no human transferrin presence and yet, the antigens may be present in a concentration lower than the cut-off value, for which the reaction will not take place with red-coloured conjugate complex. The anti-human transferrin antibodies present on the membrane (test line) will not capture the antigen-red-coloured conjugate complex (not formed), for which the red line will not appear.

Whether the sample is positive or not, the mixture continues to move across the membrane to the immobilized specific antibodies placed in the control line. The anti-specific protein antibodies present on the membrane will capture control green-conjugate complex and the control line will always appear. The presence of this green line serves as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents.

STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The test is stable until the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

PRECAUTIONS

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contamination errors.
- The tests should be discarded in a proper biohazard container after testing.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provided in the kit are approved for use in the CerTest Transferrin test. Do not use any other commercial kit component.

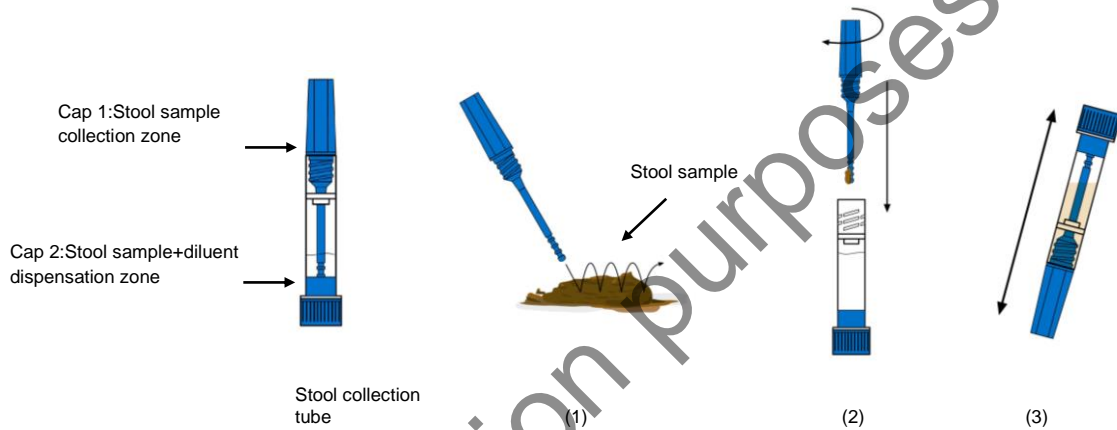
- The presence of yellow lines in the results window (control line zone and test line zone), before using the test, is completely normal and does not imply failure of the test functionality.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.

SPECIMEN COLLECTION AND PREPARATION

Stool samples should be collected in clean containers. The samples can be stored in the refrigerator (2-8°C) for 7 days prior to testing. For longer storage, maximum 6 months, the specimen must be kept frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool samples as thoroughly as possible prior to preparation.

Specimen preparation (see illustration):

1. Take out the cap 1 of the stool collection tube and use the stick to pick up sufficient sample quantity. Introduce the stick once into 4 different parts of the stool sample (1); making sure that, in each insertion, the stick's screw is covered with the sample to take the appropriate amount of stool. Then, introduce the sample in the stool collection tube (2). For liquid samples, add approx. 20µL in the stool collection tube using a micropipette.
1. Close the tube with the diluent and stool sample (2). Shake the tube in order to assure good sample dispersion (3). The stool collection tube with dilute sample can be stored until 2 days at room temperature or until 5 days in the refrigerator (2-8°C) prior to testing.

Precautions: Patients should not collect samples during their menstrual period if they have bleeding hemorrhoids, blood in urine or if they have strained during bowel movement.

MATERIALS

MATERIALS PROVIDED

- CerTest Transferrin card tests
- Instructions for use
- Stool collection tubes with diluent

MATERIALS REQUIRED BUT NO PROVIDED

- Specimen collection container
- Disposable gloves
- Timer

TEST PROCEDURE

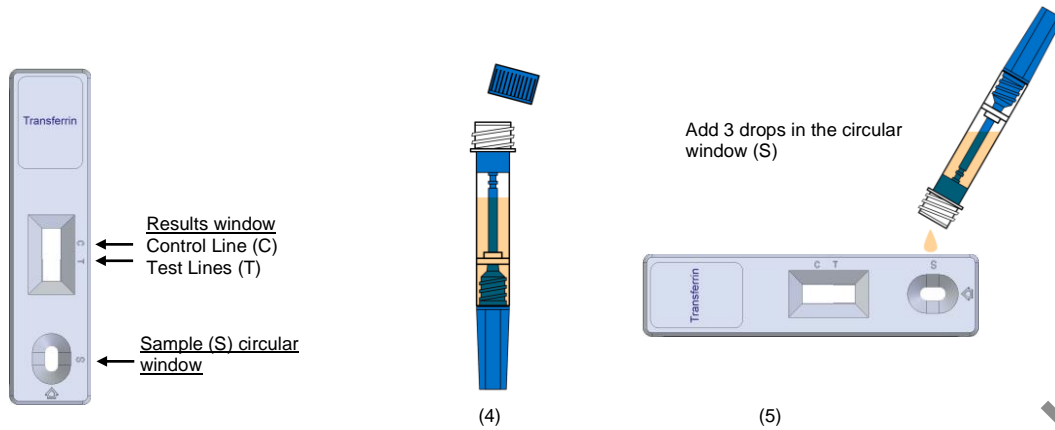
Allow tests, stool samples and controls to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Proceed to shake the stool collection tube in order to assure good sample dispersion.
2. Remove the CerTest Transferrin card test from its sealed bag just before using it.
3. Take the stool collection tube, open the cap 2 (4) and dispense 3 drops in the circular window marked with the letter S (5). Avoid adding solid particles with the liquid.
4. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes.


If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.

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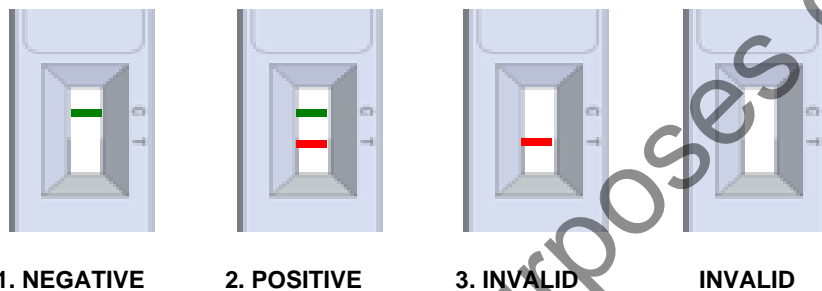INTERPRETATION OF RESULTS (please refer to the illustration below)

	Transferrin (hTf)	Interpretation of the results
1.	- GREEN	There is no human transferrin presence. Transferrin marker is not present in patient sample (<4ng/mL for transferrin), which might mean no faecal occult blood and no gastrointestinal bleeding.
2.	+ GREEN-RED	There is human transferrin presence. That probably would mean a lower gastrointestinal bleeding disease (colorectal cancer). However, it should not discriminate upper gastrointestinal disease.
3.	Any other result	Invalid result, we recommend repeating the assay using the same sample with another test.

NEGATIVE: Only the GREEN line appears across the results window in the site marked with the letter C (control line).

POSITIVE: In addition to the GREEN control line, a RED line (test line) also appears in the site marked with the letter T (test line) in the results window.

INVALID: Total absence of the control coloured line (GREEN) regardless the appearance or not of the test line (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persist, discontinue using the test kit and contact your local distributor.

NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured band in the test line (T) in the results window will vary depending on the concentration of human transferrin present in the specimen.

QUALITY CONTROL

Internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

LIMITATIONS

1. The test must be carried out within 2 hours after opening the sealed bag.
2. An excess of sample could cause wrong results (brown bands appear). Dilute the sample with the diluent and repeat the test.
3. The intensity of test lines may vary from very strong at high human transferrin concentration to faint when the human transferrin concentration is close to the cut-off value of the test.

4. CerTest Transferrin should be used only with samples from human faeces. The use of other samples has not been established. The quality of the test depends on the quality of the sample; proper faecal specimens must be obtained.
5. Positive results determine the presence of human transferrin in faecal samples; nevertheless, it can be due to several causes, besides colorectal bleeding, such as haemorrhoids, blood in urine or stomach irritations. A positive result should be followed up with additional diagnostic procedures to determine the exact cause and source of the blood.
6. A negative result is not meaningful because of it is possible the human transferrin concentration in the stool sample is lower than cut-off values. Negative results do not exclude bleeding, as some polyps and colorectal cancers may bleed intermittently or not during certain stages of the disease. Moreover, blood may not be uniformly distributed in stool samples.
7. Patients should not collect samples during their menstrual period if they have bleeding haemorrhoids, blood in urine or if they have strained during bowel movement.
8. Mucous stool samples may cause non-specific reactions in the test. Every mucous stool sample whose result is positive should be followed up with other techniques of diagnosis to confirm the result.
9. Faecal samples whose antigen concentration is near to the cut-off value of the CerTest Transferrin test could be lost if the sample has been diluted and preserved until 2 days at room temperature or until 5 days at 2-8°C until to be tested. For faecal samples near to the cut-off value is better to dilute the sample and make immediately the test.

EXPECTED VALUES

Colorectal cancer affects both men and women of all racial and ethnic groups, being most often found in people aged 50 years or older. Regarding men, colorectal cancer is the third most common cancer after prostate and lung cancers. Regarding women, colorectal cancer is the third most common cancer after breast and lung cancers.

Some studies established equal or higher 0.4 µg hTf/g faeces for transferrin as cut-off value to allow detect adult patients with gastrointestinal bleeding that will require to diagnosis additional diagnostic invasive procedures.

PERFORMANCE CHARACTERISTICS

Analytical sensitivity

Cut-off value of CerTest Transferrin is 4ng/mL (0.4µg hTf/g faeces) for human transferrin.

Clinical Sensitivity and Specificity

An evaluation, with faecal samples, was performed comparing the results obtained by an immunochromatographic test (CerTest Transferrin, CerTest) other commercial ELISA test (Human Transferrin ELISA kit, ICL). The results were as follows:

Data for cut-off value 0.4 µg hTf/g

		ELISA test: Human Transferrin ELISA kit		
		+	-	Total
IC test: CerTest Transferrin	+	38	3	41
	-	12	47	59
Total		50	50	100

CerTest Transferrin vs Human Transferrin ELISA kit		
	Mean Value	95% CI (confidence interval)
Sensitivity	76.0%	61.8 – 86.9%
Specificity	94.0%	83.5 – 98.7%
PPV	92.7%	80.1 - 98.5%
NPV	79.7%	67.2 –98.0%

Sensitivity for a cut-off value of 2 µg hTf/g: 81.1% (mean value); 64.8-92.0% (95% confidence interval).

Specificity for a cut-off value of 0.2 µg hTf/g: 96.3% (mean value); 81.0-99.9% (95% confidence interval).

The results showed a high sensitivity and specificity to detect human transferrin using CerTest Transferrin.

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Cross reactivity

An evaluation was performed to determine the cross reactivity of CerTest Transferrin; no cross reactivity against other gastrointestinal pathogens, other organism, substances and/or faecal markers occasionally present in faeces:

<i>Adenovirus</i>	<i>Cryptosporidium</i>	<i>Human lactoferrin</i>	<i>Salmonella typhimurium</i>
<i>Astrovirus</i>	<i>Entamoeba dispar</i>	<i>Legionella pneumophila</i>	<i>Shigella boydii</i>
<i>Bovine haemoglobin</i>	<i>Entamoeba histolytica</i>	<i>Listeria monocytogenes</i>	<i>Shigella dysenteriae</i>
<i>Bovine Transferrin</i>	<i>Escherichia coli O111</i>	<i>Norovirus GI</i>	<i>Shigella flexneri</i>
<i>Campylobacter coli</i>	<i>Escherichia coli O26</i>	<i>Norovirus GII</i>	<i>Shigella sonnei</i>
<i>Campylobacter jejuni</i>	<i>Escherichia coli O157</i>	<i>Pig haemoglobin</i>	<i>Streptococcus pneumococcal</i>
<i>Clostridium difficile antigen GDH</i>	<i>Giardia</i>	<i>Rotavirus</i>	<i>Streptococcus pyogenes</i>
<i>Clostridium difficile Toxin A</i>	<i>Helicobacter pylori</i>	<i>Salmonella enteritidis</i>	<i>Yersinia enterocolitica O:3</i>
<i>Clostridium difficile Toxin B</i>	<i>Human calprotectin</i>	<i>Salmonella paratyphi A</i>	<i>Yersinia enterocolitica O:9</i>
<i>Clostridium perfringens</i>	<i>Human haemoglobin</i>	<i>Salmonella typhi</i>	

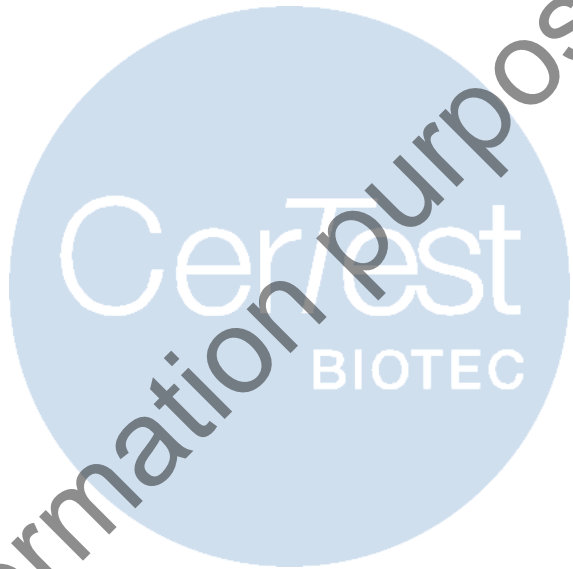
No special diet is recommended prior to testing. There are not interferences with any foods (vitamin C, broccoli, carrots...).

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		ELISA test: Human Transferrin ELISA kit		
		+	-	Total
IC test: CerTest Transferrin	+	38	3	41
	-	12	47	59
	Total	50	50	100

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1. Towler BP, Irwig L, Glasziou P, Weller D, Kewenter J. "Screening for colorectal cancer using the faecal occult blood test, Hemoccult". Cochrane Database Syst Rev. 2000;(2): CD001216.
2. Ransohoff DF and Lang CA. "Screening for colorectal cancer with the Fecal Occult Blood Test: a background paper. Ann Intern Med". 1997; 126: 811-822.
3. Ransohoff DF and Lang CA. "Suggested technique for Faecal Occult Blood testing and interpretation in colorectal cancer screening". Ann Intern Med. 1997; 126: 808-810.
4. Chien-Hua Chiang et al., "A Comparative Study of Three Fecal Occult Blood Test in Upper Gastrointestinal Bleeding". Kaohsiung J. Med. Sci. May 2006, 22; 223-228.
5. Virtanen et al., "Higher concentrations of serum transferrin receptor in children than in adults", Am. J. Clin. Nutr. 1999; 69: 256-60.

SYMBOLS FOR IVD COMPONENTS AND REAGENTS/SÍMBOLOS PARA REACTIVOS Y PRODUCTOS PARA DIAGNÓSTICO IN VITRO

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