

## ETV6/NTRK3 gene fusion t(12; 15) probe reagent Instructions Manual

**[Product Name]** ETV6/NTRK3 gene fusion t(12; 15) probe reagent

**[Package Specifications]** 10 Tests/box

**[Intended use]**

The reagent carries out in situ hybridization staining on the basis of routine staining to provide doctors with auxiliary information for diagnosis. The test results are only for clinical reference and should not be used as the only basis for clinical diagnosis. Clinicians should comprehensively judge the test results in combination with the patient's condition, drug indications, treatment response and other laboratory test indicators.

**[Detection principle]**

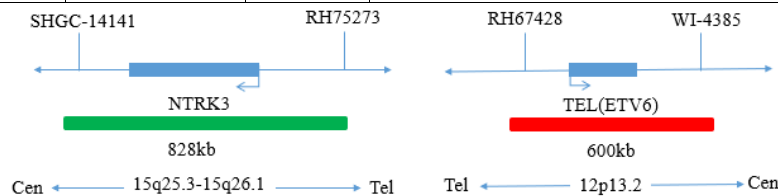
Fluorescence in situ hybridization is a technique for directly observing specific nucleic acids in cells in vitro. According to the principle of base complementary pairing, the specific probe is complementary to the target sequence in the cell. Due to the fluorescence of the probe, the gene state of the hybrid probe and the target sequence can be clearly observed under the fluorescence microscope under the appropriate excitation light.

**[Product Main Components]**

The kit consists of ETV6/NTRK3 dual-color probe, as shown in Table 1.

Table 1 Kit composition

Component name	Specifications	Quantity	Main components
ETV6/NTRK3 dual-color probe	100µl/Tube	1	ETV6 Orange probe, NTRK3 Green probe, hybridization buffer



**[Storage conditions & Validity]**

Keep sealed away from light at  $-20^{\circ}\text{C}\pm 5^{\circ}\text{C}$ , and the validity period is 12 months.

After the cover is opened, it can be sealed and stored in  $2\sim 8^{\circ}\text{C}$  away from light within 24 hours. After the cover is opened, it should be sealed and stored in  $-20\pm 5^{\circ}\text{C}$  away from light for a long time. Transport with temperature below  $0^{\circ}\text{C}$ .

**[Applicable Instruments]**

Fluorescence microscopy imaging systems, including fluorescence microscopy and filter sets suitable for DAPI (367/452), Green (495/517), and Orange (547/565).

**[Sample Requirements]**

1. Applicable specimen types: Paraffin-embedded specimens for surgical resection or biopsy.
2. Tissue should be fixed with 4% neutral formaldehyde fixation solution within 1 hour after in vitro, and the tissue should be fixed by conventional dehydration and paraffin embedding.

**[Test method]**

### 1. Pre-hybridization or Pretreatment

It is recommended to use Wuhan HealthCare Biotechnology Co., Ltd.'s "FISH Pretreatment Reagent " (Cat# CL-003) for pretreatment.

### 2. Denaturation and Hybridization

The following operations need to be carried out in the darkroom.

- ① Take out the probe, leave it at room temperature for 5min, turn it upside down with force, mix it well, and then centrifuge it for a short time (no vortex instrument vibration). Take 10µL of it and drop it into the cell drop hybridization area, immediately cover the cover glass of 22mm × 22mm. The probe should be evenly expanded under the cover glass without bubbles, and seal the edge with rubber glue (the edge must be completely sealed to prevent the dry piece from affecting the test results in the hybridization process).
- ② The cell drops were placed on the hybridizer and denatured at  $85^{\circ}\text{C}$  for 5 min (the hybridizer should be preheated to  $85^{\circ}\text{C}$ ) and hybridized at  $42^{\circ}\text{C}$  for 2-16 hours.

### 3. Washing

The following operations need to be carried out in the darkroom.

- ① Carefully remove the sealing glue around the cover glass with tweezers to avoid sticking or moving the cover glass, immerse the sample in 2xSSC for about 5S, take it out, gently push a corner of the cover glass to the edge of the slide with tweezers, and gently remove the cover glass with tweezers;
- ② Place the sample at 2xSSC room temperature for 1 min;
- ③ Take out the sample and immerse it in 0.3%NP-40/0.4xSSC solution preheated at 68°C for 2min;
- ④ Take out the sample and immerse it in deionized water preheated at 37°C in advance for 1 min; dry it naturally in the dark place.

#### 4. Counterstaining

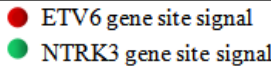

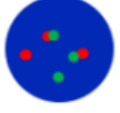
The following operations should be performed in a darkroom

10µl DAPI compound dye is dropped in the hybridization area of the glass slide and immediately covered. The suitable filter is selected for glass slide observation under the fluorescence microscope.

#### 5. FISH results observation

Place the counterstained film under the fluorescence microscope, and first put it under the low-power objective lens (10 ×) Confirm the cell area under the microscope; Go to 40× Under the objective lens, find a position where the cells are evenly distributed; Then in the high-power objective (100×) The FISH results of nuclei were observed.

#### [Common Signal Type Interpretation]

	
	Negative: 2 orange 2 green
	Positive : 1 orange 1 green 2 fusion

#### [Limitations of test methods]

- ① The results of this kit will be affected by various factors of the sample itself, but also limited by hybridization temperature and time, operating environment and the limitations of current molecular biology technology, which may lead to wrong results.
- ② Users must understand the potential errors and accuracy limitations that may exist in the detection process.

#### [Precautions]

1. This product is only used for in vitro diagnosis.
2. Please read this manual carefully before testing. The testing personnel shall receive professional technical training, and the signal counting personnel must be able to observe and distinguish orange and green signals.
3. When testing clinical samples, when the hybridization signal counting is difficult and the sample is not enough to repeat the retest, or the cell volume is not enough for analysis, the test will not provide the test results.
4. DAPI counterstaining agent used in this experiment has potential toxicity or carcinogenicity, so it is necessary to operate in the fume hood, wear masks and gloves to avoid direct contact.
5. All chemicals are potentially dangerous. Avoid direct contact. Used kits are clinical waste and should be properly disposed of.

#### [Basic information]

Name of registrant / manufacturer: Wuhan HealthCare Biotechnology Co., Ltd.

Address: Floor 1-4, Building #8, Optics Valley Precision Medicine Industry Base Phase I, #9 Gaokeyuan 3rd Road, East Lake High-Tech Zone, Wuhan City, Hubei Province, People's Republic of China.

Contact: 027-87570662

Name of after-sales service unit: Wuhan HealthCare Biotechnology Co., Ltd.

Contact: 18140559890

Production address: Floor 1-4, Building #8, Optics Valley Precision Medicine Industry Base Phase I, #9 Gaokeyuan 3rd Road, East Lake High-Tech Zone, Wuhan City, Hubei Province, People's Republic of China.

#### [Medical device registration certificate No. / product technical requirement No.]

Medical device registration certificate No.: ehxb20190983

**[approval date and modification date of the specification]**

V1. 0 approval date: November 4, 2019

V1. 2 approval date: December 7, 2021