

Single-Use Tourniquet

NEW!



The hygienic solution for improved safety

- Minimises risk of contamination
- Easy handling
- No pinching – pain-free also on hairy skin


Optimal hygienic standard

Single-use tourniquets minimise cross-contamination hazards and nosocomial infections caused by venipuncture.


This tourniquet is made from a special composite material that provides an optimal hygienic standard along with maximum patient comfort.




User Guide

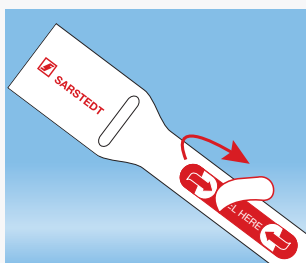


- Pull tourniquet from the packaging and tear off at the perforated line
- Remove label "PEEL HERE"

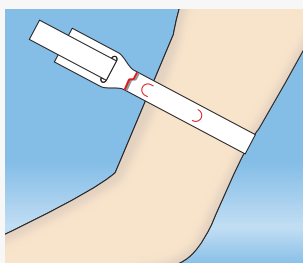

- Pass tourniquet around arm with the printed side up

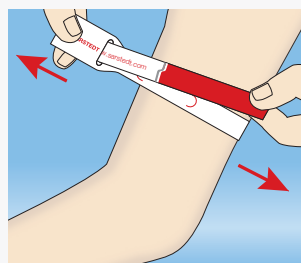


- Fasten a few inches above the puncture site and tape in place
- The pulse must be perceptible (50 - 100 Hg tourniquet pressure)
- Maximum constriction time: 1 min

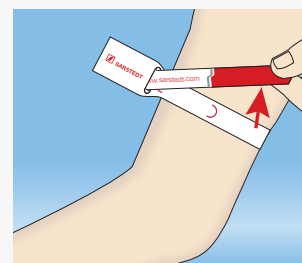

- Peel off and dispose

Technical modifications reserved

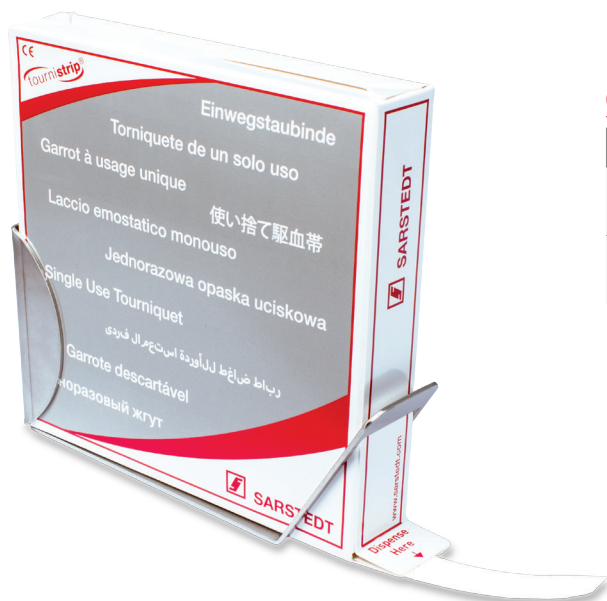

Ordering information

Order No.	Description	Packaging
95.1006	Single-use tourniquet	200/inner box
95.1005	Wall holder	1/inner box

This document may contain information on products that may not be available in particular countries

20-685-0100-200

SARSTEDT AG & Co. KG
 P.O. Box 12 20
 D-51582 Nümbrecht
 Phone +49 2293 305 - 0
 Fax +49 2293 305 - 3992
 export@sarstedt.com
 www.sarstedt.com