

Safety-Lancet – The new generation

Safe and easy sampling with increased patient comfort



Safety for both user and patient

- The needle or blade is safely concealed in the lancet body before and after use
- Needlestick injuries and cross-contamination are prevented
- The sterile single-use product cannot be re-used


Easy sampling

- Primed and ready for use
- Secure firing button prevents accidental activation and inactivation of the lancet system
- Ridged lancet body ensures safe grip
- Small contact face for precise puncturing

Patient comfort

- Smooth siliconised blades and triple-sharpened needlepoints to minimise pain
- Defined penetration depths for optimal sampling from adults and children
- No danger of bone injuries due to minimal penetration depths

Safety-Lancet – The new generation

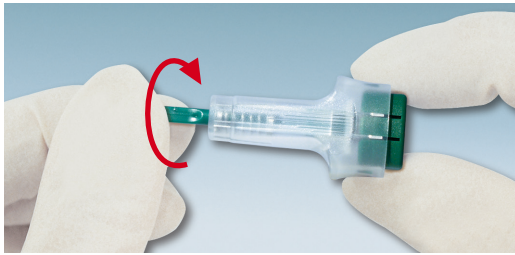

- 1 Twist the cap until it separates from the Safety-Lancet.



- 2 Press the Safety-Lancet firmly against the chosen and cleaned puncture site and press the firing button.



- 3 Dispose of the Safety-Lancet into a sharps container.

Ordering information

Order No.	Description	Size	Penetration depth	Packaging
85.1015	Safety-Lancet Mini	28 G	1.6 mm	200/inner box 1,200/case
85.1016	Safety-Lancet Normal	21 G	1.8 mm	200/inner box 1,200/case
85.1017	Safety-Lancet Extra	18 G	1.8 mm	200/inner box 1,200/case
85.1018	Safety-Lancet Super	Blade 1.5 mm	1.6 mm	200/inner box 1,200/case
85.1019 (For infants' heels)	Safety-Lancet Neonatal	Blade 1.5 mm	1.2 mm	200/inner box 1,200/case

Microvette® capillary blood collection tubes complement this product range



Microvette® 100/200
Blood collection with
End-to-End capillary



Microvette® 300/500
Blood collection with
collection rim

More information is available on request.

SARSTEDT AG & Co. KG
P.O. Box 12 20
D-51582 Nümbrecht
Phone: +49 2293 305 0
Fax: +49 2293 305 3992
export@sarstedt.com
www.sarstedt.com